

Application Sheet

XDRAFT Application

REF 51275

**Instrument:
Elisys Quattro**

REF 16300

Setting: XDRAFT_anti-HCV_51275_R5

For all essential product information, please refer to the User Manual of the analyser and the Instructions for Use for reagents, controls and calibrators. Please refer to the Instructions for Use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyser.

Applications marked with the prefix XDRAFT in the filename must be understood as DRAFT, which means that it is a proposal by HUMAN that has not been fully validated. According to HUMAN's internal SOPs, only a general run was performed and the validation criteria were met.

The user is responsible for the validation of the prepared protocols.

Material Required

Material	REF	Comments
anti HCV	51275	
The Working Solutions have to be prepared according to the instructions for use.		
WCON 2, SUB prepared in	17350/B25	Reagent bottle (25 ml)

Additional Notes

Elisys Quattro:

OD-values of 3.5 or above are outside the photometer's dynamic range. For this reason, they are displayed as 4.0.