

Application Sheet

HSV 1 IgG

REF 51216

Instrument: Elisys Uno

REF 17350

Setting:

HSV_G_51216_R4

For all essential product information, please refer to the User Manual of the analyser and the Instructions for Use for reagents, controls and calibrators. Please refer to the Instruction for Use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyser.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Material Required

Material	REF	Comments
HSV 1 IgG	51216	
Reagent Rack, Infect. Diseases & Steroids	17359/2	all reagents
Sample Rack (13 mm) Elisys Uno or Sample Rack (12 mm) Elisys Uno	17359/8 17359/7	Sample and dilution, according to sample collection tube
Sample vials 2 ml	17350/16	Sample dilution

Additional Notes

For technical reasons the assay step „start incubation time” requires to define a corresponding reagent. This reagent only exists in the software, there is no reagent bottle for it. However, this virtual reagent needs to be allocated to a position on the reagent rack, any position can be chosen. Do not place any other reagent bottle on this position while the virtual bottle is allocated to it.

Due to the usable space on the racks the number of samples that can be processed is limited to 48.