

Application Sheet

Rubella IgM μ -capture

REF 51108

Instrument: HumaReader Single plus

REF 18000

Setting:

RUM μ _51108_R5.asy

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls, calibrators and the Certificate of Analysis. Please refer to the instruction for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Additional Notes

The validity criterion "MPC/COV > 2" must always be checked by the user.

The criteria for PC, CC and Blank are checked automatically. The reading may be cancelled automatically because of invalid results for PC and/or CC, while blank values above the validity criterion do not terminate the reading.