

Application Sheet

IMTEC-t-Transglutaminase-Antibodies IgG

REF ITC30300

Instrument: HumaReader Single plus

REF 18000

Setting:

tTGG ITC30300 R1

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls, calibrators and the Certificate of Analysis. Please refer to the instructions for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Additional Notes

All validation criteria described in the IFU must be checked manually by the user.

Each assay run must be performed with a new calibration curve. When the reading gets started a dialogue asks if the calibrator values from the previous reading should be used. Select "Discard" to continue.