

REF	Content tests	Test code	Method	Units	R = ready to use P = reconst.	Chimney	Calibrator	Calibration points	Calibration type	on board Stability (days)	Calibration stability (days)	Measuring range	Sample Type*	Required solutions	HumaTrol N /P	SERODOS / SERODOS ^{plus}	TURBIDOS	Other controls	Reference ranges (adults) m = male f = female
ACID PHOSPHATASE																			
10660600	3 x 50	ACP	a-Naphtyl phosphate, Hillmann mod.	U/l; µkat/l	P		AUTOCAL	1	Linear	7	7	2.4 - 50 U/l	S	DIL	X	X	–		m: <6.6 U/l; <0.11 µkat/l f: <6.5 U/l; <0.11 µkat/l
ALBUMIN liquicolor																			
10560300	4 x 60	ALB	Bromcresol green	g/dl; g/l	R		AUTOCAL	1	Linear	56	35	0.5 – 8.0 g/dl; 5 – 80 g/l	S; EDTA; HEP	DIL	X	X	–		≤60 years: 3.5–5.3 g/dl; 35–53 g/l
10560600	6 x 150									56	35								
ALKALINE PHOSPHATASE liquicolor																			
12117300	2 x 150	ALP	AMP buffer, 37°C, IFCC	U/l; µkat/l	R	R1	AUTOCAL	1	Linear	28	20	10 - 2000 U/l; 0.17 - 33.3 µkat/l	S; HEP	DIL	X	X	–		m: 40 - 130 U/l; 0.67 - 2.17 µkat/l f: 44 - 105 U/l; 0.92 - 1.75 µkat/l
12117600	3 x 250									28	20								
ALKALINE PHOSPHATASE opt. liquicolor																			
12027300	2 x 150	ALP-2	DEA buffer, 37°C, GSCC/DGKC	U/l; µkat/l	R	R1	AUTOCAL	1	Linear	28	28	10 - 2000 U/l; 0.17 - 33.3 µkat/l	S; HEP	DIL	X	X	–		m: 80 - 306 U/l; 1.33 - 5.10 µkat/l f: 64 - 306 U/l; 1.07 - 5.10 µkat/l
12027600	3 x 250									28	28								
alpha-AMYLASE liquicolor																			
12028300	2 x 100	AMY	CNPG3, 37°C, IFCC	U/l; µkat/l	R		AUTOCAL	1	Linear	56	56	10 - 4500 U/l; 0.17 - 75 µkat/l	S; HEP	DIL	X	X	–		≤220 U/l; ≤3.67 µkat/l
12028600	3 x 260									56	56								
anti-STREPTOLYSIN O																			
11251300	1 x 100	ASO	Immunoturbidimetry, Latex-enhanced	IU/ml; kIU/l	R ⁴⁾	R1	ASO Standard	1	Linear	49	28	10 – 500 IU/ml; 10 – 500 kIU/l	S	DIL, SWA-2	–	–	X		≤200 IU/ml; ≤200kIU/l
11251600	1 x 210									49	28								
auto-BILIRUBIN-D liquicolor																			
10741300	2 x 100	DBIL	DPD method	mg/dl; µmol/l	R		AUTOCAL	1	Linear	28	28	0.1 - 9.0 mg/dl; 1.71 - 154 µmol/l	S; HEP	DIL	X	X	–		<0.2 mg/dl; <3.42 µmol/l
10741600	5 x 210									28	28								
auto-BILIRUBIN-T liquicolor																			
10742300	2 x 150	TBIL	DPD method	mg/dl; µmol/l	R		AUTOCAL	1	Linear	56	30	0.20 - 20 mg/dl; 3.42 - 342 µmol/l	S; HEP	DIL	X	X	–		0.1 - 1.2 mg/dl; 2 - 21 µmol/l
10742300	5 x 210									56	30								

1) after separation from cells or stabilized plasma, e.g. sodium fluoride, mannose, 2) except ammonia heparinate, 3) mix gently before place on the analyzer, 4) mix R2 gently before place on the analyzer, 5) LOT-specific range
*Sample Type: S = Serum, P = Plasma, EDTA, HEP = Heparin plasma, CITRATE, Urine, CSF = Cerebrospinal fluid

REF	Content tests	Test code	Method	Units	R = ready to use P = reconst.	Chimney	Calibrator	Calibration points	Calibration type	on board Stability (days)	Calibration stability (days)	Measuring range	Sample Type*	Required solutions	HumaTrol N /P	SERODOS / SERODOS ^{plus}	TURBIDOS	Other controls	Reference ranges (adults) m = male f = female
auto-CREATININE liquicolor																			
10052300	2 x 150	CREA	Jaffé	mg/dl;	R	R1	AUTOCAL	1	Linear	21	7	0.10 - 25.0 mg/dl;	S; HEP	DIL	X	X	–		m: 0.70 - 1.30 mg/dl; 62 - 115 µmol/l
10052600	5 x 250			µmol/l						21	7	8.84 - 2210 µmol/l							f: 0.58 - 1.12 mg/dl; 51 - 99 µmol/l
CALCIUM liquicolor																			
10011300	3 x 100	Ca	Ortho-cresolphthalein	mg/dl;	R	R1	AUTOCAL	1	Linear	21	1	0.25 - 20 mg/dl;	S; HEP	DIL	X	X	–		8.6 - 10.3 mg/dl; 2.15 - 2.58 mmol/l
10011600	3 x 430			mmol/l						21	1	0.06 - 5.00 mmol/l							
CHOLESTEROL liquicolor																			
10028300	2 x 100	CHOL	CHOD PAP	mg/dl;	R		AUTOCAL	1	Linear	56	56	5 - 1100 mg/dl;	S; EDTA;	DIL	X	X	–		≤190 mg/dl; ≤5.0 mmol/l
10028600	3 x 150			mmol/l						56	56	0.13 - 28.4 mmol/l	HEP						
CHOLINESTERASE liquicolor																			
12007300	2 x 100	CHE	Butyrylthiocholine, 37°C, DGKC	U/l; kU/l; µkat/l	R		AUTOCAL	1	Linear	42	42	500 – 20000 U/l (0.5 – 20 kU/l); 8 – 333 µkat/l	S	DIL		X			m: 4620 - 11500 U/l; 77 - 195 µkat/l f: 3930 - 10800 U/l; 65 - 180 µkat/l
CK NAC liquiUV																			
12015600	2 x 180	CK	Enzymatic, 37°C, IFCC	U/l; µkat/l	R		AUTOCAL	1	Linear	56	56	7 – 2000 U/l; 0.12 – 33.3 µkat/l	S; HEP	DIL	X	X	–		m: ≤ 190 U/l; ≤ 3.20 µkat/l f: ≤ 170 U/l; ≤ 2.85 µkat/l
CK-MB liquiUV																			
12118600	2 x 100	CKMB	Immuno inhibition	U/l; µkat/l	R		CK-MB Calibrator	1	Linear	56	56	5 - 2000 U/l; 0.08 - 33.3 µkat/l	S; Li-HEP	DIL	–	–	–	CK-MB Control	<24 U/l; <0.4 µkat/l
CHLORIDE liquicolor																			
10115300	2 x 50	Cl	TPTZ method	mmol/l	R		AUTOCAL	1	Linear	35	14	7 – 500 mmol/l; 25 – 1773 mg/dl	S; HEP; CSF	DIL-2	X	X	–		Serum: 95 - 105 mmol/l; 337 - 383 mg/dl CSF: 116 - 127 mmol/l; 411 - 450 mg/dl
COMPLEMENT C3																			
11110600	1 x 70	C3	Immunoturbidimetry	mg/dl; g/l	R	R1	C3/C4/TRF Standard	6	Five parameters	63	63	5 - 350 mg/dl 0.05 - 3.50 g/l	S	DIL, C-CLEAN	–	–	X		90 - 180 mg/dl; 0.9 - 1.8 g/l
COMPLEMENT C4																			
11113600	1 x 70	C4	Immunoturbidimetry	mg/dl; g/l	R	R1	C3/C4/TRF Standard	6	Five parameters	63	63	4 - 60 mg/dl 0.04 - 0.60 g/l	S	DIL, C-CLEAN	–	–	X		20 - 50 mg/dl; 0.2 - 0.5 g/l

1) after separation from cells or stabilized plasma, e.g. sodium fluoride, mannose, 2) except ammonia heparinate, 3) mix gently before place on the analyzer, 4) mix R2 gently before place on the analyzer, 5) LOT-specific range
*Sample Type, S = Serum, P = Plasma, EDTA, HEP = Heparin plasma, CITRATE, Urine, CSF = Cerebrospinal fluid

REF	Content tests	Test code	Method	Units	R = ready to use P = reconst.	Chimney	Calibrator	Calibration points	Calibration type	on board Stability (days)	Calibration stability (days)	Measuring range	Sample Type*	Required solutions	HumaTrol N /P	SERODOS / SERODOS ^{plus}	TURBIDOS	Other controls	Reference ranges (adults) m = male f = female
CREATININE (enzym) liquicolor																			
10053300	1 x 95	CREA-2	Enzymatic	mg/dl;	R		AUTOCAL	1	Linear	56	56	0.15 – 40.0 mg/dl; 13.3 – 3536 µmol/l	S; HEP	DIL	X	X	–		m: 0.64 - 1.19 mg/dl; 57 - 105 µmol/l f: 0.40 - 1.00 mg/dl; 35 - 88 µmol/l
10053600	1 x 275			µmol/l						56	56								
CRP																			
11241300	1 x 100	CRP	Immunoturbidimetry	mg/dl;	R	R2	CRP Standard	6	Poly-linear	49	49	0.3 - 16.0 mg/dl; 3.0 - 160 mg/l	S; EDTA; HEP	DIL, C-CLEAN	–	–	X		≤0.5 mg/dl; ≤5 mg/l
11241600	1 x 250			mg/l						49	49								
gamma-GT liquicolor																			
12023300	2 x 100	GGT	gamma-Glutamyl-3carboxy-4-nitroanilide, 37°C, IFCC	U/l;	R		AUTOCAL	1	Linear	56	56	4 - 1300 U/l; 0.07 - 21.7 µkat/l	S; EDTA; HEP	DIL	X	X	–		m: <60 U/l; <1.0 µkat/l f: <40 U/l; <0.65 µkat/l
12023600	3 x 250			µkat/l						56	56								
GLUCOSE liquicolor																			
10260300	2 x 150	GLUC	GOD	mg/dl;	R		AUTOCAL	1	Linear	56	56	2 - 500 mg/dl; 0.11 - 27.8 mmol/l	S; EDTA; HEP ¹⁾	DIL	X	X	–		60 - 95 mg/dl*; 3.3 - 5.3 mmol/l* *fasting
10260600	6 x 210			mmol/l						56	56								
GOT (ASAT) IFCC mod. liquiUV																			
12021300	2 x 200	GOT	Enzymatic, 37°C, IFCC without P5P	U/l; µkat/l	R		AUTOCAL	1	Linear	56	56	3 - 500 U/l; 0.05 - 8.33 µkat/l	S; EDTA; HEP	DIL	X	X	–		m: <35 U/l; <0.58 µkat/l f: <31 U/l; <0.52 µkat/l
12021600	5 x 310			µkat/l						56	56								
GPT (ALAT) IFCC mod. liquiUV																			
12022300	2 x 200	GPT	Enzymatic, 37°C, IFCC without P5P	U/l; µkat/l	R		AUTOCAL	1	Linear	56	56	7.3 - 440 U/l; 0.12 - 7.33 µkat/l	S; EDTA; HEP	DIL	X	X	–		m: <45 U/l; <0.74 µkat/l f: <34 U/l; <0.56 µkat/l
12022600	5 x 310			µkat/l						56	56								
HbA1c liquidirect																			
10770600	1 x 135	HbA1c	Immunologic assay	% HbA1c; mmol/mol Hb	R ³⁾	R1	HbA1c liquidirect Calibrator Set	5	Four parameters	28	14	15 - 140 mmol/mol Hb ⁵⁾ ; 3.5 - 15% HbA1c ⁵⁾	whole blood (EDTA)	DIL, C-CLEAN	–	–	–	HbA1c liquidirect Control Set	<6% HbA1c; <42 mmol/mol Hb
HDL CHOLESTEROL liquicolor																			
10084300	2 x 100	HDL-C	Homogenous enzymatic assay	mg/dl;	R		AUTOCAL	1	Linear	21	14	5 - 150 mg/dl; 0.13 - 3.88 mmol/l	S; EDTA; HEP	DIL	–	X	–		>60 mg/dl; >1.54 mmol/l
10084600	2 x 180			mmol/l						14	14								

1) after separation from cells or stabilized plasma, e.g. sodium fluoride, mannose, 2) except ammonia heparinate, 3) mix gently before place on the analyzer, 4) mix R2 gently before place on the analyzer, 5) LOT-specific range
*Sample Type, S = Serum, P = Plasma, EDTA, HEP = Heparin plasma, CITRATE, Urine, CSF = Cerebrospinal fluid

REF	Content tests	Test code	Method	Units	R = ready to use P = reconst.	Chimney	Calibrator	Calibration points	Calibration type	on board Stability (days)	Calibration stability (days)	Measuring range	Sample Type*	Required solutions	HumaTrol N /P	SERODOS / SERODOS ^{plus}	TURBIDOS	Other controls	Reference ranges (adults) m = male f = female
IMMUNOGLOBULINS direct IgA																			
11501300	1 x 100	IgA	Immunoturbidimetry	mg/dl; g/l	R	R2	Immunoglobulins direct Calibrator Set	6	Four param- eters	42	42	5 - 600 mg/dl	S	DIL, C-CLEAN	-	X	X	0	70 - 500 mg/dl; 0.70 - 5.0 g/l
11501600	1 x 250									42	42								
IMMUNOGLOBULINS direct IgG																			
11502300	1 x 100	IgG	Immunoturbidimetry	mg/dl; g/l	R	R2	Immunoglobulins direct Calibrator Set	6	Four param- eters	42	42	26 - 2600 mg/dl	S	DIL, C-CLEAN	-	X	X	0	700 - 1600 mg/dl; 7.0 - 16.0 g/l
11502600	1 x 250									42	42								
IMMUNOGLOBULINS direct IgM																			
11503300	1 x 100	IgM	Immunoturbidimetry	mg/dl; g/l	R	R2	Immunoglobulins direct Calibrator Set	6	Four param- eters	42	42	7 - 395 mg/dl	S	DIL, C-CLEAN	-	X	X	0	m: 40 - 230 mg/dl; 0.4 - 2.3 g/l f: 40 - 280 mg/dl; 0.4 - 2.8 g/l
11503600	1 x 250									42	42								
IRON TPTZ liquicolor																			
12290300	2 x 100	Fe	TPTZ	µg/dl; µmol/l	R		AUTOCAL	1	Linear	56	30	10 - 1000 µg/dl; 2 - 179 µmol/l	S; HEP	DIL	X	X	-		m: 59 - 158 µg/dl; 10.6 - 28.3 µmol/l f: 37 - 145 µg/dl; 6.6 - 26.0 µmol/l
12290600	2 x 180									30	30								
LDH SCE mod. liquiUV																			
12014300	2 x 100	LDH-P	Pyruvate, 37°C, SCE	U/l; µkat/l	R		AUTOCAL	1	Linear	56	56	15 - 1600 U/l; 0.25 - 26.7 µkat/l	S; EDTA; HEP	DIL	X	X	-		225 - 450 U/l; 3.75 - 7.50 µkat/l
12014600	2 x 180									56	56								
LDL CHOLESTEROL liquicolor																			
10094600	2 x 90	LDL-C	Homogenous enzymatic assay	mg/dl; mmol/l	R		AUTOCAL	1	Linear	56	14	5 - 300 mg/dl; 0.13 - 7.76 mmol/l	S; EDTA; HEP	DIL	-	X	-		≤100 mg/dl; ≤2.59 mmol/l
LIPASE liquicolor																			
12006600	2 x 100	LIP	Methylresorufin, 37°C	U/l; µkat/l	R		AUTOCAL	1	Linear	21	21	2 - 300 U/l; 0.03 - 5.0 µkat/l	S; HEP	DIL	-	X	-		13 - 60 U/l; 0.2 - 1 µkat/l

1) after separation from cells or stabilized plasma, e.g. sodium fluoride, mannose, 2) except ammonia heparinate, 3) mix gently before place on the analyzer, 4) mix R2 gently before place on the analyzer, 5) LOT-specific range
*Sample Type, S = Serum, P = Plasma, EDTA, HEP = Heparin plasma, CITRATE, Urine, CSF = Cerebrospinal fluid

REF	Content tests	Test code	Method	Units	R = ready to use P = reconst.	Chimney	Calibrator	Calibration points	Calibration type	on board Stability (days)	Calibration stability (days)	Measuring range	Sample Type*	Required solutions	HumaTrol N /P	SERODOS / SERODOS ^{plus}	TURBIDOS	Other controls	Reference ranges (adults) m = male f = female
MAGNESIUM liquicolor																			
10010300	4 x 30	Mg	Xylidyl blue	mg/dl; mmol/l	R	R1	AUTOCAL	1	Linear	2	1	0.30 – 4.80 mg/dl; 0.13 – 2.00 mmol/l	S; HEP	DIL	X	X	–		1.7 - 2.6 mg/dl; 0.70 - 1.05 mmol/l
10010600	4 x 60									3	1								
MICROALBUMIN																			
11120600	1 x 90	MALB	Immunoturbidimetry	mg/l	R	R2	MICROALBUMIN Standard	1	Linear	49	30	5 - 400 mg/l5)	collected urine; spot urine	DIL, C-CLEAN	–	–	–		m: <30 mg/24h; <30 mg/g Crea; ≤ 2.5 mg/mmol Crea f: <30 mg/24h; <30 mg/g Crea; ≤ 3.5 mg/mmol Crea
PANCREAS-AMYLASE liquicolor																			
12009600	2 x 100	PAMY	EPS-G7, 37°C	U/l; µkat/l	R		AUTOCAL	1	Linear	56	56	5 - 2300 U/l; 0.08 - 38.3 µkat/l	S; EDTA; HEP	DIL	X	X	–		<53 U/l; <0.88 µkat/l
PHOSPHORUS liquirapid																			
10027300	4 x 30	PHOS	Molybdate (UV)	mg/dl; mmol/l	R		AUTOCAL	1	Linear	60	60	1.1 – 18.3 mg/dl; 0.36 – 5.91 mmol/l	S	DIL	X	X	–		2.6 - 4.5 mg/dl; 0.84 - 1.45 mmol/l
10027600	4 x 60									60	60								
POTASSIUM liquiUV																			
10120300	1 x 120	K	Enzymatic	mmol/l	R		POTASSIUM liquiUV Standard	2	Linear	56	7	2 – 8 mmol/l	S; Li-HEP	DIL-2	X	X	–		3.6 - 5.5 mmol/l
10120600	1 x 250									56	7								
RHEUMATOID FACTORS																			
11261300	1 x 100	RF-2	Immunoturbidimetry, Latex-enhanced	IU/ml; kIU/l	R		RF Standard	4	Poly-linear	90	14	15 - 100 IU/ml	S	DIL, SWA-2	–	–	X		≤20 IU/ml; ≤20 kIU/l
11261600	1 x 210									90	14								
SODIUM liquicolor																			
10113300	1 x 120	Na	Enzymatic	mmol/l	R		SODIUM liquicolor Standard	2	Linear	56	7	90 – 180 mmol/l	S; Li-HEP	DIL-2	X	X	–		135 - 145 mmol/l
10113600	1 x 250									56	7								
TOTAL PROTEIN liquicolor																			
10570300	2 x 150	TP	Biuret	g/dl; g/l	R	R1	AUTOCAL	1	Linear	28	28	0.2 - 12 g/dl; 2 - 120 g/l	S; EDTA; HEP	DIL	X	X	–		6.6 - 8.3 g/dl; 66 - 83 g/l
10570600	6 x 210									28	28								

1) after separation from cells or stabilized plasma, e.g. sodium fluorid, mannose, 2) except ammonia heparinate, 3) mix gently before place on the analyzer, 4) mix R2 gently before place on the analyzer, 5) LOT-specific range
*Sample Type, S = Serum, P = Plasma, EDTA, HEP = Heparin plasma, CITRATE, Urine, CSF = Cerebrospinal fluid

REF	Content tests	Test code	Method	Units	R = ready to use P = reconst.	Chimney	Calibrator	Calibration points	Calibration type	on board Stability (days)	Calibration stability (days)	Measuring range	Sample Type*	Required solutions	HumaTrol N /P	SERODOS / SERODOS ^{plus}	TURBIDOS	Other controls	Reference ranges (adults) m = male f = female
TRANSFERRIN																			
11115600	1 x 70	TRF	Immunoturbidimetry	mg/dl; g/l	R	R1	C3/C4/TRF Standard	6	Four parameters	63	63	3 – 850 mg/dl ⁵⁾ or 0.03 – 8.50 g/l ⁵⁾	S	DIL, C-CLEAN	–	–	X		170 - 340 mg/dl; 1.70 - 3.4 g/l
TRIGLYCERIDES liquicolor^{mono}																			
10724300	3 x 60	TG	GPO PAP	mg/dl;	R		AUTOCAL	1	Linear	56	56	10 - 1000 mg/dl; 0.11 - 11.4 mmol/l	S; EDTA; HEP	DIL	X	X	–		≤150 mg/dl; ≤1.71 mmol/l
10724600	3 x 210			mmol/l						56	56								
UREA liquiUV																			
10521300	2 x 150	UREA	Urease (UV)	mg/dl;	R		AUTOCAL	1	Linear	56	56	5 - 350 mg/dl; 0.83 - 58.3 µmol/l	S; EDTA; HEP ²⁾	DIL	X	X	–		m: 18 - 55 mg/dl; 3.0 - 9.2 mmol/l f: 15 - 43 mg/dl; 2.6 - 7.2 mmol/l
10521600	5 x 250			mmol/l						56	56								
URIC ACID liquicolor plus																			
10694300	2 x 100	UA	Uricase	mg/dl;	R		AUTOCAL	1	Linear	56	24	1 - 25 mg/dl; 59.5 - 1487 µmol/l	S; EDTA; HEP	DIL	X	X	–		m: 3.6 - 7.0 mg/dl; 214 - 416 µmol/l f: 2.3 - 6.0 mg/dl; 137 - 357 µmol/l
10694600	5 x 250			µmol/l						56	24								
URINARY TOTAL PROTEIN liquicolor																			
10590300	3 x 50	UTP	Pyrogallol red	mg/l; mg/dl	R		URINARY TOTAL PROTEIN Standard	1	Linear	56	28	50 – 2000 mg/l; 5 – 200 mg/dl	Urine; CSF	DIL, SWA-2	–	–	–		Urine: 24 - 141 mg/day CSF: < 500 mg/l; < 50 mg/dl

1) after separation from cells or stabilized plasma, e.g. sodium fluorid, mannose, 2) except ammonia heparinate, 3) mix gently before place on the analyzer, 4) mix R2 gently before place on the analyzer, 5) LOT-specific range
*Sample Type, S = Serum, P = Plasma, EDTA, HEP = Heparin plasma, CITRATE, Urine, CSF = Cerebrospinal fluid